

ORDER OF DRAW CHART



LID COLOUR		COLLECTION TUBE	INVERT TO MIX
Aerobic	Grey flip off lid	Blood Cultures	N/A
Anaerobic	Purple flip off lid		
	Light blue	Citrate Tube	3 - 4 times
	Bright yellow	CPD Tube	8 - 10 times
	Red	Plain Serum Tube (with clot activator)	5 - 6 times
	Gold	SST™II Gel Tube	6 times
	Green	Heparin Tube	8 – 10 times
Nil / Negative control	Grey	Quantiferon-TB Gold	Shake 10 times
TB1 antigens	Green		Shake 10 times
TB2 Antigens	Yellow		Shake 10 times
Mitogen tube positive control	Purple		Shake 10 times
	Pink	EDTA Tube	8 - 10 times
	Purple	EDTA Tube	8 – 10 times
	Navy Blue	(EDTA) Trace Element	8 – 10 times
	Fawn/Black	Cell Free DNA BCT	8 – 10 times
	Grey	Fluoride Oxalate and Fluoride EDTA Tubes	8 - 10 times